

The design of our models is protected against counterfeiting. ^{TS}

Nursing blanket 40052^{TS} CE

The nursing blanket is a more human restraint for restless residents. It is ideal for both user and nurse/caregiver. It gives the user sufficient freedom of movement and replaces unpleasant restraints. It also prevents exposure and has a preventive effect on hygiene.

The residents can easily be changed by means of the zipper while they are in the blanket. The material is very strong, has a high tear resistance and is flame-retardant. The nursing blanket can be considered as a sheet, with a built-in jacket. It is closed by a sturdy zipper that closes towards the foot end.

Quality 1010: Breathable, moisture-repellent and flame-retardant fabric with a high tear resistance.

It is REQUIRED to always use the nursing blanket together with a bed rail.



Md 40052^{TS}

Mattress-encompassing nursing blanket, long sleeves, neck with zip shielding, 2 side openings with zipper, front zipper.

Optional: *40052 C : 10 cm extra height

	S/M	L/XL	XXL
Chest circumference (cm)	<85	85-125	>125

Washing instructions for quality 1010:

1. Washing Wash separately at 60°C with little detergent.
2. Rinse DON'T ADD FABRIC CONDITIONER.
3. Dry Dry short at low temperature in the drier or dry hanging.
Don't dry together with hydrophilic materials (such as cotton,...).

The design of our models is protected against counterfeiting. ^{TS}



1. Place the bottom layer under the mattress and attach the upper part to the bottom layer with the dark grey zips.



2. Open the light grey zipper at the front and let the patient sit on the mattress. Insert both arms through the sleeves and close the zipper.



3. Close the protective strip with the plastic snap fasteners and raise the side rails into high position.