

## **Sensory- and weighted articles**

The design of our models is protected against counterfeiting. TS

# 40332: Flame-retardant washable weighted blanket Anne-Lore® (granulate)

Weighted articles are used as a soothing aid for people who are subject to stress and anxiety related factors or for people with sleeping problems.

A weighted blanket soothes in the same way as a hug would do. Studies of the use of weighted articles have shown that for some patients they reduce the need for restraints and medication.

A weighted article filled with granules perfectly follows the contours of the body. It also has the advantage that you don't need a lot of filling volume to apply enough pressure, so that the article itself doesn't have to be really thick. This is practical in a warm environment and easy to maintain.







**Md 40332/4**<sup>TS</sup>: Weighted blanket in qual. 1010, dim. 140/200 cm, 4 kg, press stud ribbon on the sides. **Md 40332/6**<sup>TS</sup>: Weighted blanket in qual. 1010, dim. 140/200 cm, 6 kg, press stud ribbon on the sides.

Md 40332/8<sup>TS</sup>: Weighted blanket in qual. 1010, dim. 140/200 cm, 8 kg, divisible by a zipper in the middle, press stud ribbon on the sides.

Md 40332/10<sup>TS</sup>: Weighted blanket in qual. 1010, dim. 140/200 cm, 10 kg, divisible by a zipper in the middle, press stud ribbon on the sides.

Md 40332/12<sup>TS</sup>: Weighted blanket in qual. 1010, dim. 140/200 cm, 12 kg, divisible by a zipper in the middle, press stud ribbon on the sides.

### Purpose:

- Reduce anxiety and stress
- Relieve restlessness and irritability
- > Improve concentration

#### **Attention:**

- The combination of a weighted blanket and a reduced lung capacity or a weak musculature may involve a risk. In this case, follow the use of the blankets closely, or consult a doctor if in doubt.
- Consider the risk of tripping if a weighted blanket has fallen to the floor.





# **Sensory- and weighted articles**

The design of our models is protected against counterfeiting. TS

#### **Guidelines for use:**

The weighted blanket 40332 is best used in combination with duvet cover 610510 A. To do this, open the duvet cover's zip and turn it inside out. Press the press stud ribbon on the sides together so that both parts hang together. Turn it all back through the zip opening and finally close the zip.











The zipper in the middle of blanket 40332 is to split the blanket into 2 parts of lower weight for washing.

Quality: 1010, flame-retardant, breathable material, high tear resistance

Stuffing: flame-retardant plastic granulate

The weighted articles are made of a flame-retardant fabric.

## Washing instructions for weighted blankets:

1. Washing Separate washing at 60°C gentle programme with a <u>little</u> detergent in a washing bag.

Duvets weighing more than 6 kg are divisible so that each part can be washed

separately.

2. Rinse DON'T ADD FABRIC CONDITIONER.

3. Dry Dry very briefly at a low temperature in a laundry bag in the dryer or hang to dry.

Do not dry together with moisture-retaining textiles (like cotton,...).

