

The design of our models is protected against counterfeiting. ^{TS}

Nursing blanket Md 40049^{TS}

The nursing blanket is a more human restraint for restless residents. It is ideal for both user and nurse/caregiver. It gives the user sufficient freedom of movement and replaces unpleasant restraints. It also prevents exposure and has a preventive effect on hygiene.

The residents can easily be changed by means of the zipper while they are in the blanket. The material is very strong, has a high tear resistance and is flame-retardant. The nursing blanket can be considered as a sheet, with a built-in jacket. It is closed by a sturdy zipper that closes towards the foot end.

Quality 1010: Breathable, moisture-repellent and flame-retardant fabric with a high tear resistance.

It is REQUIRED that the nursing blanket always be used together with a bed rail.



Md 40049^{TS}

Mattress-encompassing nursing blanket, (dark grey zipper around the mattress), 2 side openings with zipper (for probes, light grey), no front zipper, zipper over the shoulder to the side and till the feet (light grey zipper), 6x strap with a buckle to attach the nursing blanket to the bed frame. (size S/M, **L/XL**, XXL).

Optional: *40082 : 10 cm extra height

	S/M	L/XL	XXL
Chest circumference (cm)	<85	85-125	>125

Washing instructions for quality 1010:

1. *Washing* Wash separately at 60°C with little detergent.
2. *Rinse* DO NOT ADD FABRIC CONDITIONER.
3. *Dry* Dry very briefly at low temperature in the dryer or hang to dry.
Do not dry together with moisture-retaining textiles (like cotton,...).

The design of our models is protected against counterfeiting. ^{TS}



1. Attach underlay to upper part with 2 dark grey zippers.



2. Attach the 6 grey straps with metal buckle to the bed frame.



3. Open the nursing blanket with the light grey zipper.



4. Put the patient in bed and insert the right arm through the armhole of the jacket.



5. Insert the patient's left arm through the armhole and close the light grey zipper.

