

The design of our models is protected against counterfeiting.^{TS}

Side rail protector

The side rail protector is used to prevent restless residents from injuring themselves at the side rail.

Quality 1010: Breathable, moisture-repellent and flame-retardant textile with high tear resistance.



Md 40055^{TS}

The side rail protector is fitted with buckles and belts to the side rail and even under the mattress, so the entire bed is shielded. The side rail can be lifted without removing the protector. The fabric is soft and very strong. The filling is also flame retardant. The material is not air-proof and breathable, so there is no risk of suffocation.

This product is also available in special versions. It can be provided with openings to use catheters,...

Purpose:

- To prevent the patient from injuring himself at the side rail or by inserting limbs through the side rail or between the side rail and the bed frame.
- Avoid crawling under the siderail.
- To be used for mentally confused persons who are not aggressive.

Quality:

1010, permanently flame retardant, breathable material,
Drying cabinet resistant. Short drying time at low temperature.

Washing instructions for quality 1010 :

1. *Washing* Wash separately at 60°C with little detergent.
2. *Rinse* DO NOT ADD FABRIC CONDITIONER.
3. *Dry* Dry very briefly at low temperature in the dryer or hang to dry.
Do not dry together with moisture-retaining textiles (like cotton,...).

The design of our models is protected against counterfeiting.^{TS}

Nursing aids

