

*The design of our models is protected against counterfeiting.<sup>TS</sup>*

## Side rail protector C €

*The side rail protector is used to prevent restless residents from injuring themselves at the side rail.*

*Quality 1010: Breathable, flame-retardant textile with high tear resistance.*



### **Md 40060<sup>TS</sup>**

The side rail protector with a split in the middle (for 2 half side rails at each side), is fitted over the side rail using buckles and is also connected under the mattress so the entire bed is shielded. The side rail can be raised or lowered without having to remove the side rail protector. The fabric is soft, very strong, has a high tear resistance and is flame retardant. The filling is also flame retardant. The material is not air-proof and breathable, so there is no risk of suffocation.

#### *Purpose:*

- Prevent the patient from injuring himself at the side rail or by inserting limbs through the side rail or between the side rail and the bed frame.
- Avoid crawling under the siderail.
- To be used for mentally confused persons who are not aggressive.

#### *Quality:*

1010, permanently fire retardant, breathable material,  
Drying cabinet resistant. Short drying time at low temperature.

#### Washing instructions for quality 1010 :

1. *Washing* Wash separately at 60°C with little detergent.
2. *Rinse* DON'T ADD FABRIC CONDITIONER.
3. *Dry* Dry short at low temperature in the drier or dry hanging. Don't dry together with hydrophilic materials (such as cotton,...).

*The design of our models is protected against counterfeiting.<sup>TS</sup>*

## Nursing aids

