

Sensory- and weighted articles

The design of our models is protected against counterfeiting. $^{\mbox{\scriptsize TS}}$

40331: Flame-retardant weighted lap blanket Anne-Lore[®] (granulate)

Weighted articles are used as a soothing aid for people who are subject to stress and anxiety related factors or for people with sleeping problems. A weighted blanket soothes in the same way as a hug would do. Studies of the use of weighted articles have shown that for some patients they reduce the need for restraints and medication.

A weighted article filled with granules perfectly follows the contours of the body. It also has the advantage that you don't need a lot of filling volume to apply enough pressure, so that the article itself doesn't have to be really thick. This is practical in a warm environment and easy to maintain.



Md 40331/3^{TS}: Day blanket in qual. 1010, colour: blue, dim. 125/70 cm, 3 kg

Purpose:

- Reduce anxiety and stress
- Relieve restlessness and irritability
- Improve concentration

Attention:

- The combination of a weighted blanket and a reduced lung capacity or a weak musculature may involve a risk. In this case, follow the use of the blankets closely, or consult a doctor if in doubt.
- Consider the risk of tripping if a weighted blanket has fallen to the floor.

Guidelines for use:

Lay the lap blanket over the legs in length or width depending on the posture in the seat. Fold the blanket in half for more local pressure.

Quality :	1010, flame-retardant, breathable material, high tear resistance
Stuffing:	flame-retardant plastic granulate

Van Herck byba | Tel 0032(0) 14-58.84.80 | Fax 0032(0) 14-58.09.50 | www.van-herck.be | mail: anne-lore@van-herck.be



Md 40331/4,5^{TS:} Day blanket in qual. 1010, colour: green, dim. 125/70 cm, 4,5 kg.



The design of our models is protected against counterfeiting. ^{T5}

The weighted articles are made of a flame-retardant fabric.

Washing instr	uctions for weighted blankets:	ł
1. Washing	Separate washing at 60°C gentle programme with a little detergent in a washing bag.	ł
	Duvets weighing more than 6 kg are divisible so that each part can be washed separately.	
2. Rinse	DON'T ADD FABRIC CONDITIONER.	ł
3. Dry	Dry <u>very briefly</u> at a <u>low</u> temperature in a laundry bag in the dryer or hang to dry. Do not dry together with moisture-retaining textiles (like cotton,).	

