

The design of our models is protected against counterfeiting.^{TS}

Sensory articles

Sensory articles are used to confirm body awareness and existence of the individual in persons with dementia by giving them as many sensory stimuli as possible.



** Colours and designs may differ (squeeze figure not included, sold separately **31801**)*

Md 31842^{TS}

Sensory lap blanket, in qual. 1010, washable, with belt, sewn in balls, bells and buttons, a ring to attach something personal, Close-up® zipper, press stud tape on the inside (can be combined with **31843** PU weighted blanket).

Purpose:

- Tactile stimulation.

Guideline for use:

Place the experience blanket on the thighs and tie the belt with a simple knot on the user's back.

Quality: 1010, permanently flame retardant, breathable material, high tear resistance.

Washing instructions for the sensory articles :

1. *Washing* Wash at 60°C.
2. *Dry* Dry very briefly at low temperature in the dryer or hang to dry.