

The design of our models is protected against counterfeiting.^{TS}

Abdominal pelvic belt

*Belt to prevent people from falling down or falling over in a chair or seat.
The pelvic belt is made of a comfortable material and can be washed at 60°C and air-dried.
Quality 1065 : Breathable, moisture-repellent and flame-retardant anti-tear material.*



Md 31920 D^{TS}

Abdominal pelvic belt with an extra Close-up[®] zipper to attach a lap blanket.

Md 31920 DC^{TS}

Lap blanket in microfleece, dimensions 80/85 cm, zippable.

Md 31920 DB^{TS}: Lap blanket in qual. 1010 dim. 80/85 cm, zippable.

Purpose:

- prevent the patient from sinking into the wheelchair or care seat
- provide support so that the patient can adopt a good sitting posture
- increase patient safety and prevent the patient from falling by immobilising in the wheelchair or care seat.
- To be used for mentally confused persons, who are not aggressive.

Quality: 1065, permanently flame retardant, breathable material,
Washable at 60°C and tumble-dryer proof. Short drying time at low temperature.

Washing instructions for quality 1010 and 1065:

1. *Washing* Wash separately at 60°C with little detergent.
2. *Rinse* DON'T ADD FABRIC CONDITIONER.
3. *Dry* Dry short at low temperature in the drier or dry hanging.
Don't dry together with hydrophilic materials (such as cotton,...).