

The design of our models is protected against counterfeiting. ^{TS}

Nursing blanket 40050^{TS} CE

The nursing blanket is a more human restraint for restless residents. It is ideal for both user and nurse/caregiver. It gives the user sufficient freedom of movement and replaces unpleasant restraints. It also prevents exposure and has a preventive effect on hygiene.

The residents can easily be changed by means of the zipper while they are in the blanket. The material is very strong, has a high tear resistance and is flame-retardant. The nursing blanket can be considered as a sheet, with a built-in jacket. It is closed by a sturdy zipper that closes towards the foot end.

Quality 1010: Breathable, moisture-repellent and flame-retardant fabric with a high tear resistance.

It is REQUIRED to always use the nursing blanket together with a bed rail.



Md 40050^{TS}

Nursing blanket as a fitted sheet around the mattress, sleeveless, (size S/M, **L/XL**, XXL), neck with zip shielding, 2 side openings with zipper, front zipper.

Optional: **40050 B : zippers on the shoulders*
**40050 ED : long sleeves*
**40050 PK: jacket in children's size*

	S/M	L/XL	XXL
Chest circumference (cm)	<85	85-125	>125


Washing instructions for quality 1010:

1. *Washing* Wash separately at 60°C with little detergent.
2. *Rinse* DON'T ADD FABRIC CONDITIONER.
3. *Dry* Dry short at low temperature in the drier or dry hanging.
Don't dry together with hydrophilic materials (such as cotton,...).

The design of our models is protected against counterfeiting. TS



1. Apply the nursing blanket around the mattress like a fitted sheet. Adjust the length of the elastic band if required with the elastic clip.

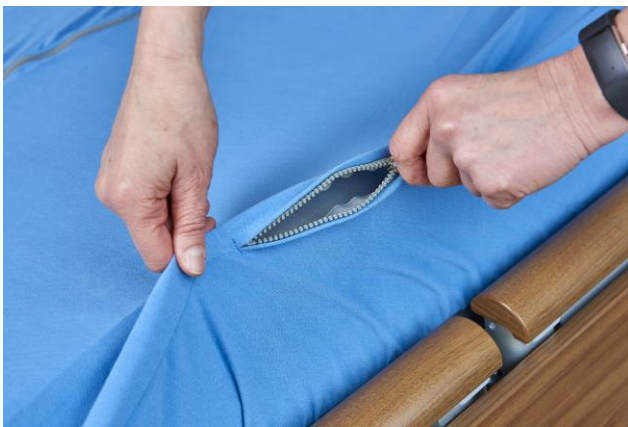

2. Close the side zippers and open the nursing blanket from the foot end to the neck opening.



3. Put the patient on the mattress, insert the arms through the armholes in the jacket and close the zipper. Close the protective strip with the plastic snap fasteners and bring the side rails into high position.

