

*The design of our models is protected against counterfeiting.<sup>TS</sup>*

### Sensory articles

*Sensory articles are used to confirm body awareness and existence of the individual in persons with dementia by giving them as many sensory stimuli as possible.*



#### Md 31843

Md 31843/3: Weighted blanket wipeable in PU, dim. 65/85 cm, 3 kg, press stud ribbon on the sides.

Md 31843/5,5: Weighted blanket wipeable in PU, dim. 65/85 cm, 5,5 kg, press stud ribbon on the sides.  
(combinable with **31842** experience lap blanket cover)

#### Purpose :

- Reduce anxiety and stress
- Alleviate unrest and irritability
- Improve concentration,...

The weighted blanket in PU is intended as a device that helps reduce anxiety and stress and improve concentration through deep pressure simulation.

#### Guidelines for use:

The weighted blanket 31843 is best used in combination with sensory cover 31842.

Open the zipper of the cover and turn it inside out. Press the press stud ribbon on the sides so that both parts hang together. Turn the whole back through the zipper opening and finally close the zipper.

Place the weighted sensory blanket on the upper legs and tie the belt with a simple knot on the back of the user.

**Quality :** PU, permanent flame retardant  
1010, permanent flame retardant, breathable material, high tear resistance.

**Filling:** flame retardant plastic granules

*The design of our models is protected against counterfeiting.<sup>TS</sup>*

### PU

1. *Washing*      *Can be cleaned with warm soap water up to 95°C.  
Don't use solvents or bleach.  
Can be chemically cleaned with all the intended liquids except trichloorethylene.*