

Sensory articles

Experience articles are used to confirm the body awareness and existence of the individual in persons with dementia by giving them as many sensory stimuli as possible.



Md 31843

Md 31843/3: Washable weighted blanket in PU, dimensions 65/85 cm, 3 kg, snap-fastener tape on the sides.

Md 31843/5.5: Washable weighted blanket in PU, dimensions 65/85 cm, 5.5 kg, snap fastener tape on the sides.

(can be combined with **31842** sensory lap blanket cover)

Purpose :

- Reduce anxiety and stress
- Alleviate restlessness and irritability
- Improve concentration
-

The weighted blanket made of PU is designed to help reduce anxiety and stress and improve concentration through deep pressure stimulation.

Instructions for use:

The weighted blanket 31843 is best used in combination with the sensory blanket cover 31842. To do this, open the zipper on the cover and turn it inside out. Press the Velcro strips on the sides together so that both parts are attached to each other. Turn the whole thing back through the zipper opening and finally close the zipper.

Place the weighted sensory blanket on the upper legs and fasten the belt with a simple knot on the user's back.

The design of our models is protected against counterfeiting. TS

Quality : PU, permanently flame-retardant
1010, permanently flame-retardant, breathable material, high tear resistance
Filling: flame-retardant plastic granules

PU

1. Cleaning Can be washed with warm soapy water up to 95°C.
Do not use solvents or bleach.
Can be dry-cleaned using any suitable cleaning solution, with the exception of trichloroethylene.