

The design of our models is protected against counterfeiting. <sup>TS</sup>

### 40333: Flame-retardant wipeable weighted blanket Anne-Lore® (granulate)

*Weighted articles are used as a soothing aid for people who are subject to stress and anxiety related factors or for people with sleeping problems.*

*A weighted blanket soothes in the same way as a hug would do. Studies of the use of weighted articles have shown that for some patients they reduce the need for restraints and medication.*

A weighted article filled with granules perfectly follows the contours of the body. It also has the advantage that you don't need a lot of filling volume to apply enough pressure, so that the article itself doesn't have to be really thick. This is practical in a warm environment and easy to maintain.



**Md 40333/4<sup>TS</sup>**: Weighted blanket wipeable in PU, dim. 140/200 cm, 4 kg, press stud ribbon on the sides.

**Md 40333/6<sup>TS</sup>**: Weighted blanket wipeable in PU, dim. 140/200 cm, 6 kg, press stud ribbon on the sides.

**Md 40333/8<sup>TS</sup>**: Weighted blanket wipeable in PU, dim. 140/200 cm, 8 kg, press stud ribbon on the sides.

**Md 40333/10<sup>TS</sup>**: Weighted blanket wipeable in PU, dim. 140/200 cm, 10 kg, press stud ribbon on the sides.

**Md 40333/12<sup>TS</sup>**: Weighted blanket wipeable in PU, dim. 140/200 cm, 12 kg, press stud ribbon on the sides.

**Md 610510 A<sup>TS</sup>** : Cover in qual. 1010, 140/200 cm, close-up<sup>®</sup> zipper, press stud ribbon on the inside.

#### Purpose:

- Reduce anxiety and stress
- Relieve restlessness and irritability
- Improve concentration

#### Attention:

- The combination of a weighted blanket and a reduced lung capacity or a weak musculature may involve a risk. In this case, follow the use of the blankets closely, or consult a doctor if in doubt.
- Consider the risk of tripping if a weighted blanket has fallen to the floor.

The design of our models is protected against counterfeiting. <sup>TS</sup>

### Guidelines for use:

The weighted blanket 40333 is best used in combination with duvet cover 610510 A. To do this, open the duvet cover's zip and turn it inside out. Press the press stud ribbon on the sides together so that both parts hang together. Turn it all back through the zip opening and finally close the zip.



*Quality:* PU, flame-retardant  
*Stuffing:* flame-retardant plastic granulate

The weighted articles are made of a flame-retardant fabric.

### Washing instructions for weighted articles in PU :

- 1. Wipe* Wipe with lukewarm soapy water or disinfect. Don't use solvents, bleaches or abrasives.