

The design of our models is protected against counterfeiting. ^{TS}

Nursing blanket 40051^{TS}

The nursing blanket is a more human restraint for restless residents. It is ideal for both user and nurse/caregiver. It gives the user sufficient freedom of movement and replaces unpleasant restraints. It also prevents exposure and has a preventive effect on hygiene.

The residents can easily be changed by means of the zipper while they are in the blanket. The material is very strong, has a high tear resistance and is flame-retardant. The nursing blanket can be considered as a sheet, with a built-in jacket. It is closed by a sturdy zipper that closes towards the foot end.

Quality 1010: Breathable, moisture-repellent and flame-retardant fabric with a high tear resistance.

It is REQUIRED that the nursing blanket always be used together with a bed rail.



Md 40051^{TS}

Mattress-encompassing nursing blanket, sleeveless, (size S/M, **L/XL**, XXL), neck with zip shielding, 2 side openings with zipper, front zipper.

Optional: *40051 B : zipper on the shoulders
 *40082 : 10 cm extra height

	S/M	L/XL	XXL
Chest circumference (cm)	<85	85-125	>125

Washing instructions for quality 1010:

1. *Washing* Wash separately at 60°C with little detergent.
2. *Rinse* DO NOT ADD FABRIC CONDITIONER.
3. *Dry* Dry very briefly at low temperature in the dryer or hang to dry.
Do not dry together with moisture-retaining textiles (like cotton,...).


The design of our models is protected against counterfeiting. ^{TS}



1. Place the lower layer under the mattress and attach the upper part to the lower layer with the dark grey zippers.



2. Open the light grey zipper at the front and let the patient sit on the mattress. Insert both arms through the armholes in the jacket and close the zipper.

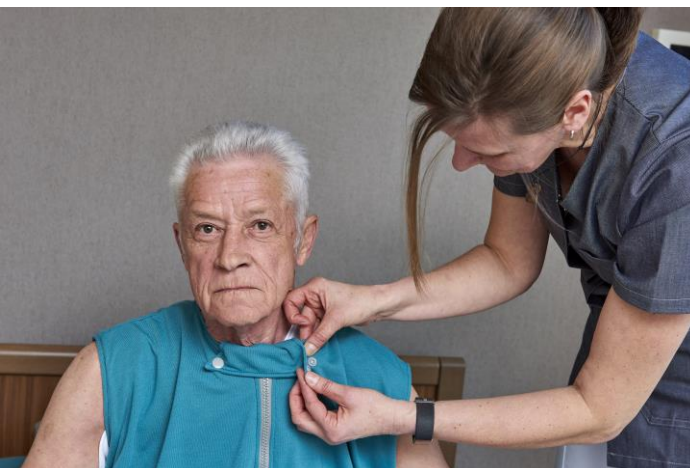

3. Close the protective strip with the plastic snap fasteners and raise the side rails into high position.